

FACE SHIELD ACP-1

Instructions for use

Safety information:


- The shield does not serve as a protective device when turning, milling, drilling, etc.
- The shield must be properly disinfected with either disinfectant or isopropyl alcohol before first and further use.
- It is not allowed to use other cleaning solutions such as acetone, petrol, toluene, etc.
- If any part of the shield is damaged, discontinue use. Dispose of it according to applicable regulations and standards.

Specifications:

- Manufactured to EN 166, PPE cat. 1, visor cat. 3.
- Shield weight approx. 80 gr.
- Sight size 310 x 250 mm (H x W).
- Visor is made of PVC

How to use our shield:

- 1.The shield must be properly disinfected with either disinfectant or isopropyl alcohol before attaching the shield.
- 2.Adjust the gray visor belt to the width of your face.
- 3.Put the shield on the head and tighten the belt on the rear buckle as required.

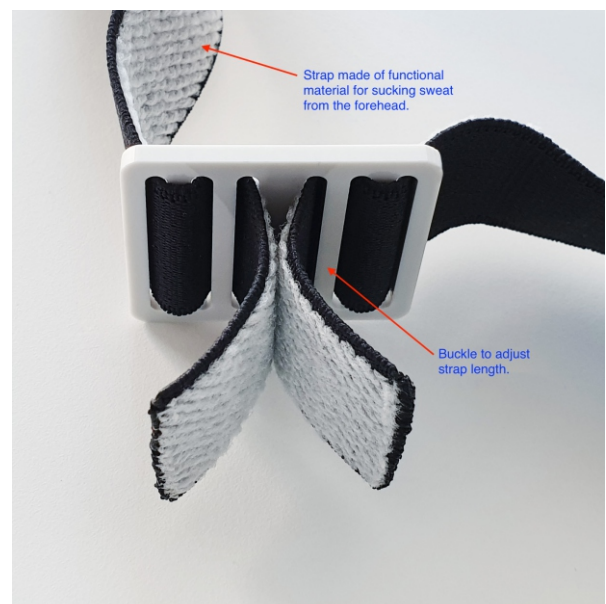

Manufacturer:

Diametral Plc.

Vaclava Spacka 1759

193 00 Praha 20

Czech Republic

tel: +420 222 360 423, mail: orders-eu@dimetral.cz